

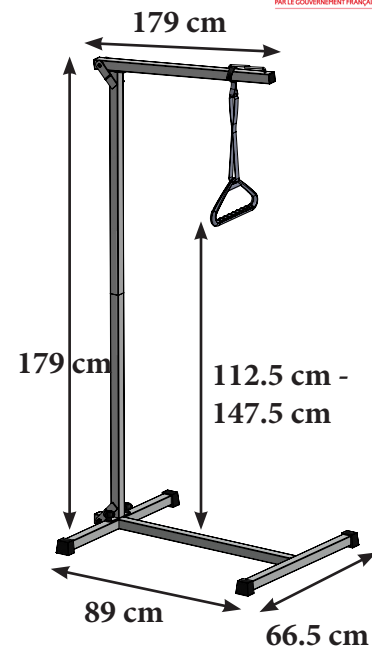
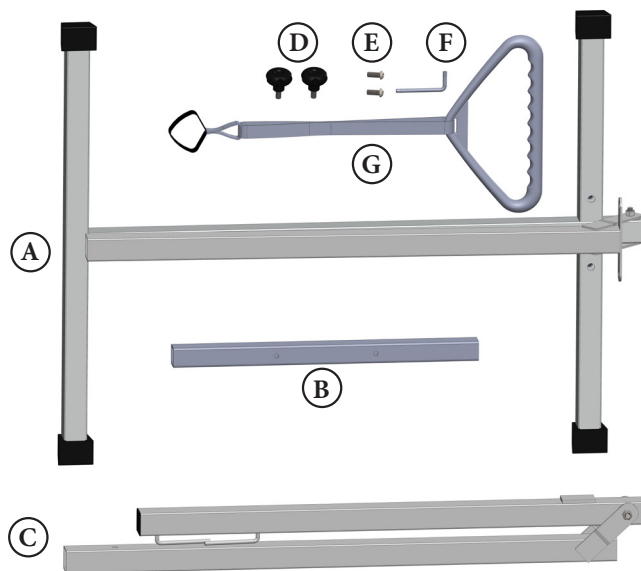
11,60 kg

130 kg

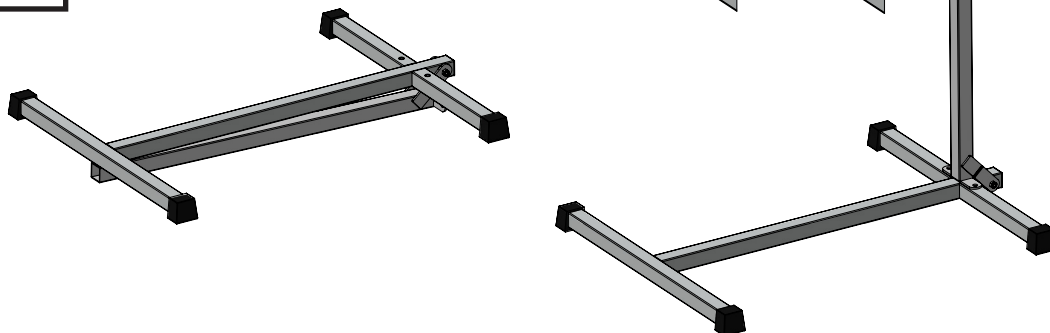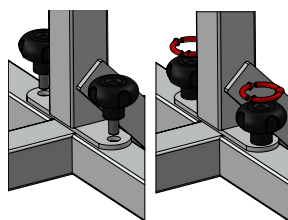1


2


3

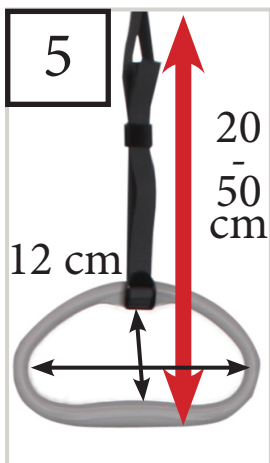4

6

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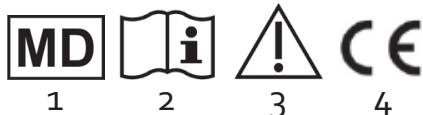**EN-** Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

### 1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0° or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, castors...).
- ♥ Check regularly if all parts are tightened.
- ♥ Use only in the intended way.
- ♥ Do not hang anything on the lifting pole.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ After assembling the product, make sure that it is securely locked in OPEN position and level to the ground before use.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by qualified technical staff.
- ♥ In case you didn't use your table for a long time, let it be checked by the technical staff, before using.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

### 2. Intended use, indications and contraindications

#### Intended use:

The lifting pole is intended to be used indoor only. It can be used with most of the usual beds and seats in order to change position. Never place the lifting pole beside the bed or chair, but always behind the back rest. Additional fixings on the wall or at the bed are not necessary.

#### Indications:

Handicap or temporary immobilisation of the patient's hip girdle, where the upper body's muscular system (shoulder girdle and upper limbs) still enables to reach a sitting position and raise the lower part of the body (in order to use a bedpan for example).

#### Contraindications:

Lifting poles may not be appropriate for people in certain circumstances, for example if they have poor trunk control or painful shoulders, for example:

- Severe cardiac problems • Severe chest problems • Upper limb girdle dysfunction e.g. Rheumatoid Arthritis, Muscular Dystrophy • Severe hemiplegia.
- Where there is any possibility of shoulder dislocation or spinal strain.
- Where there is a risk to pressure areas from potential drag effect.
- Where there are any signs of a confused state.

### 3. Description and Fitting instructions

Only specialists can adjust the product and teach you how to use it. Follow steps 1 to 4. A wrong use or adjustment could lead to overstress of the device or of the patient. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. Care should be taken to ensure that the base of the lifting pole is properly positioned under the bed, otherwise the frame may tip over. Furthermore, ensure that there is enough space between the bed / chair and the floor to slide the lifting pole under it.

#### Product description:

The lifting pole is made of a steel frame and a triangle. Both the height and the position of the handle are adjustable. The lifting pole can be disassembled quickly, which makes it easy to transport and store. This lifting pole supports up to 100kg at the trapeze handle bar, which means that the maximum user's weight is 130kg. If the product is not

used for a longer period of time it has to be stored in a dry place and protected against dust and direct sunlight.

### 4. Cleaning

The pole must be regularly cleaned with a dry and clean cloth. The trapeze handle has a date clock (7). With normal use the trapeze handle will last for five years. After 5 years, replace the trapeze handle.

### 5. Materials

Tube in chromed or epoxy painted steel, polyamide strap and PP handle.

### 6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

### 7. Technical specifications

Check the table at the end of the user's manual.

This product has a 24 months warranty, from the date of purchase, against any manufacturing defect. Duration of use: 2 years.

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